


For the Use and  
Care of

*NiteTrain-r.*

**Wet Call**

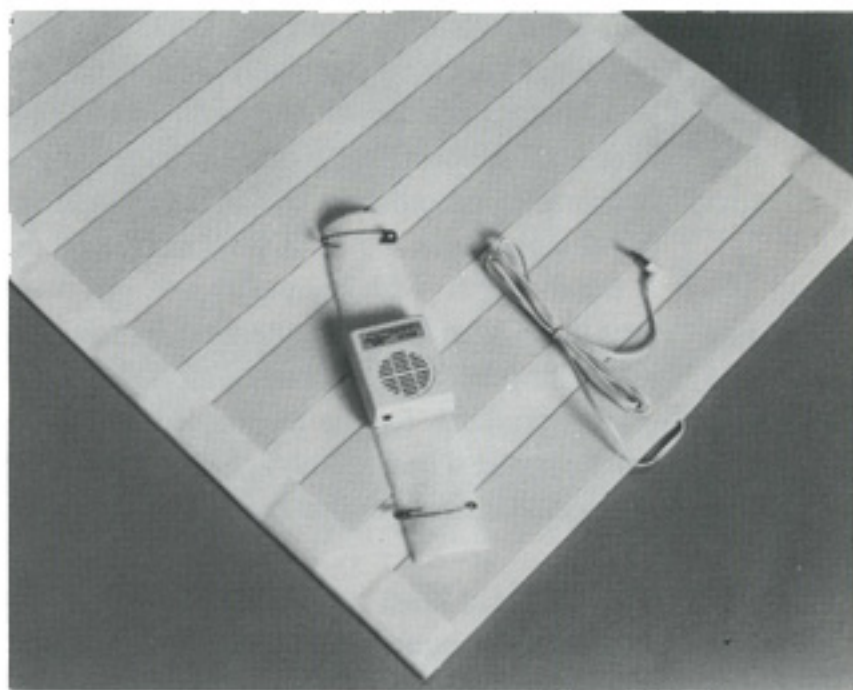

A Fast,  
Effective Way to  
**STOP**  
**BEDWETTING**

For free help or additional information, call or write:

*NiteTrain-r*<sup>®</sup>

**Wet Call**

**Manufactured by**

**Koregon Enterprises, Inc.**

9735 S.W. Sunshine Court, Suite 100

Beaverton, OR 97005

(503) 626-8833

Toll Free: 1-800-544-4240

Fax: (503) 644-1366

# Wet Call

**WHAT IS IT?** **Wet Call** consists of a small miniaturized electronic alarm box powered with a 9-volt alkaline battery and a very effective moisture sensing bed pad. The sensing pad (18-1/2" wide and 24-1/2" long) is made of a combination of Arlon polyethylene foam and Lectrolite conductive vinyl sheets. A conducting cord (66" long) is used to connect the sensing bed pad to the alarm. **Wet Call** is specifically designed for habitual bedwetters, incontinent bed patients in hospitals or nursing homes, and the physically and mentally handicapped with bedwetting or incontinence problems. **Wet Call** is a medically proven way to safely stop bedwetting and/or alert the nurse or other attendants for the physically or mentally handicapped when the bed gets wet. It is simple, comfortable, easy to use and highly effective

## **HOW TO USE**

**Wet Call** pad must be placed either in a pillow case (king or queen size) or underneath a dry bed sheet. The white sensor strips should face upward. The alarm after being connected to the bed pad can be placed either on the shoulder of the pajama top (velcro strip and two safety pins provided) or attach to the bed rail using the velcro strip. To test

the alarm, slowly insert the plug into the alarm box until a slight resistance is felt. The alarm will sound letting you know it is working. When the plug is fully inserted into the alarm box, the unit is ready to use. Nite Train'r alarm has a built-in volume control switch (high & low). Set the volume to either high or low by moving the switch located at the back of the alarm box.

## **CARE AND CLEANING**

1. Clean the pad by rinsing in warm water, wipe dry and allow to air dry. Do not soak the pad in water. Avoid using hot water or hot air. Do not machine wash. Do not use any detergent or other cleansing agents.
2. Do not get the alarm box wet.
3. When removing the pad from the bed, be sure to grasp the pad not the cord. The wires are delicate and could pull apart.
4. When you are not using **Wet Call**, pull the plug out of the alarm box. This will prolong the life of your battery.

## **WARRANTY**

Nite Train'r **Wet Call** carries a full 60-day parts and labor warranty. Beyond this time limit, the device will be repaired at our laboratory at a reasonable cost.