



T-Reader[®]

Drug of Abuse Test Reader




For Forensic Use Only

- Portable Tabletop Instrument
- Capture Results Automatically
- Eliminate Visual Interpretation
- Compatible with T-Cup/T-Cube
- Barcode Verification
- USB Cable Connection

Test Procedure


1 Place the T-Cup/T-Cube into the cup holder, be sure to match the QR code on the cup with the indicator on the cup holder.

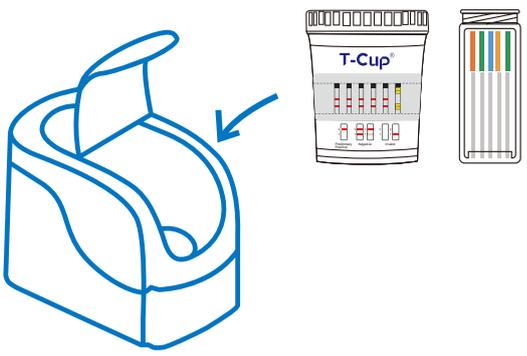


2 Open the T-Reader software, and login with user name and password.




3 Click "Read test device" in the test analysis window. The T-Reader will test automatically and display the test result on the computer screen. It normally takes 1-2 mins to read the result.

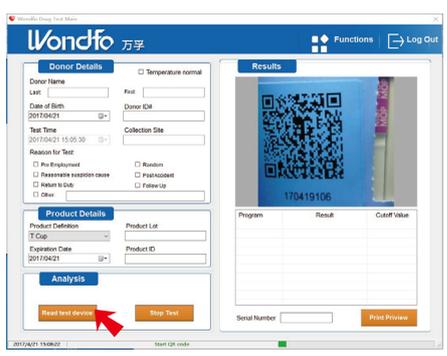

4 Click "Print Preview", an overview screen will be displayed. The operator can choose to either save the result or print it out.

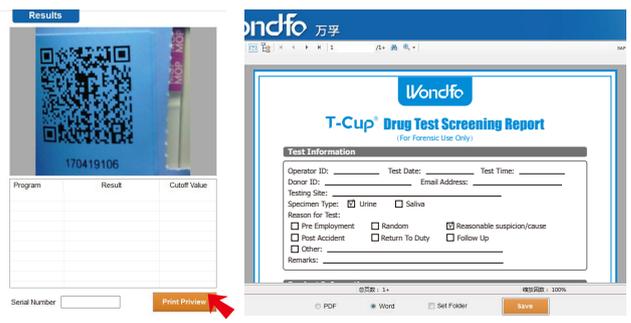

5 Remove the T-Cup/T-Cube from the T-Reader.

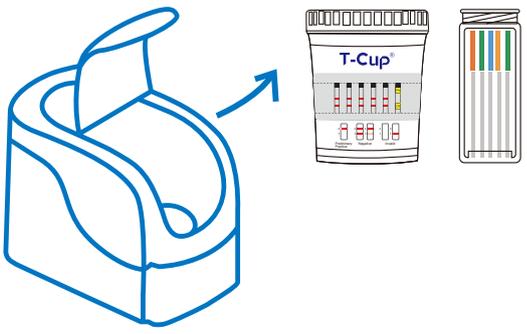

Order Information

Item Code	Description	Complexity
IP-201	T-Reader [®] DOA Test Reader	Forensic Use Only