

## PRODUCT CARE



Hand wash with mild detergent and water below 30°C without wringing.



Dry in shade on a flat surface.



Do not dry-clean.



Do not Iron.



Do not bleach.



Close hook loop fasteners before washing.

**EC** **REP**

MT Promed Consulting GmbH  
Altenhofstrasse 80  
D-66386 St. Ingbert  
Germany  
tel.: +49 (0) 6894 581020  
fax: +49 (0) 6894 581021  
e-mail: info@mt-procons.com  
www.mt-procons.com

### Symbol Explanation :



**LOT** Batch code



**M** Manufacturing date



**F** Manufacturer



**REF** Catalogue number

  
**tynor**

**Wrist Splint Ambidextrous**

रिस्ट स्प्लिन्ट एम्बिडेक्स्ट्रस  
Fijación Ambidiestra De Muñeca  
Attelle de poignet Ambidextre  
جبيرة المعصم بواجهتين

**REF E 43**




UM-E43-18-2

  
**tynor**

**Tynor Orthotics Pvt. Ltd.**

169-170, Sec 82, Ind. Area, JILP, Mohali-160 055, (Pb) India  
e-mail: marketing@tynorindia.com • www.tynorindia.com  
Customer Care : 0091-172-5288111

**ISO**  **W.H.O.** **ISO** **CE** **US FDA**  
9001 **G.M.P.** 13485 registered

[www.tynorindia.com](http://www.tynorindia.com)

USER MANUAL

## How To Use Cómo Utilizar



## कैसे उपयोग करें Comment utiliser: كيفية الاستخدام

1.



- Aluminum splint is pushed out. The contours of the splint are matched to the contours of the wrist according to the degree of dorsi flexion required.
- एल्युमिनियम की खपच्ची को बाहर धकेला जाता है। बन्धी का दांचा कलाई के ढांचे से आवश्यक ढकड के लिए डोरसी लॉचनीयता की डिग्री के अनुसार मिलाया
- Quitar la férula de aluminio. Formatear la férula en conformidad con el formato de la muñeca, según el grado de dorsiflexión necesario.
- L'attelle en aluminium est repoussée. Les contours de l'attelle sont adaptés aux contours du poignet en fonction du degré de flexion dorsale requis.

• تدفع الشريحة الألمونيوم للخارج بحيث يتوافق محيط الجبيرة مع محيط المعصم حسب درجة الانثناء الخلفي المطلوب

2.



- The customized splint is pushed back into the pocket, so that the splint comes on the palm side of the hand
- विशिष्ट रूप से बनाई गई खपच्ची को जेब में वापिस धकेल दिया जाता है, ताकि स्प्लिंट हाथ की हथेली पर आ सके।
- Volver la parte posterior de la férula al bolsillo de nylon, de forma que la férula se presenta en el lateral de la palma de la mano
- L'attelle personnalisée est repoussée dans la poche, de sorte que l'attelle se place sur le côté paume de la main

• وهكذا تصبح الجبيرة على جانب راحة اليد  
• يتم إعادة الجبيرة المخصصة داخل الجيب مجدداً.

## How To Use Cómo Utilizar



## कैसे उपयोग करें Comment utiliser: كيفية الاستخدام

3.



- The injured hand is pushed into the tubular opening of the product and is aligned with the palmer crease on the top.
- चोटिल हाथ को उत्पाद के ट्यूबलर के मुख में धकेला जाता है और शीर्ष पर पाल्मर क्रीज के साथ अलाइन किया जाता है।
- La mano lesionada se introduce en la abertura tubular del producto y se alinea con el pliegue palmar situado en la parte superior.
- La main blessée est poussée dans l'ouverture tubulaire du produit et est alignée avec le pli palmer sur le dessus.

• يتم دفع اليد المصابة في المدخل الأنبوبي للمنتج بمحاذاة تجعد راحة اليد الموجود على القمة.

4.



- Thumb is adjusted to a comfortable position in the thumb opening. The hook and loop rejoiners are fastened to a comfortable compression.
- अंगूठे को थंब ओपनिंग में आरामदायक स्थिति में समायोजित किया जाता है। हुक लूप को पुनः बांधने वाले बन्धों द्वारा सुविधाजनक संकुचन के लिए बान्ध दिया जाता है।
- El pulgar se ajusta a una posición cómoda en la abertura del pulgar. Mantenga el cierre de velcro a una compresión confortable.
- Le pouce est ajusté à une position confortable dans l'ouverture du pouce. Les suspensions crochet et boucle sont fixées à une compresion confortable.

• يتم ضبط الإبهام في وضع مريح في مدخل الإبهام.  
• يتم ضبط الإبهام إلى وضع مريح في الفتحة الإبهام.

## Wrist Splint Ambidextrous

### English

#### INTENDED USE :

Wrist Splint Ambidextrous is designed to immobilize and provide firm support to hand and wrist in various orthopedic conditions. It immobilizes the wrist in the dorsi-flexion/ functional position while allowing full range of motion to fingers and thumb. It compresses and retains the body heat to allay the local pain and inflammation. The product can be removed to permit, therapy sessions, washing and hygiene.

#### CONSTRUCTION:

Made out of an extremely dermophillic, durable, cotton based, elasticized fabric. It is shaped anatomically to provide uniform compression around the wrist. The rigid support is provided by anatomically contoured aluminum splints. These splints can be molded in cold to fit the individual patient accurately. Hook loop closures help in easy application and adjustment of the product.

The thumb apertures are designed so that the same product can be used for both left and the right hand just by swapping the splint from one pocket to another. It maintains the thumb in its abduction position.

#### APPLICATIONS:

- Early cast removal
- Fracture management
- Post surgical protection and rehabilitation
- Rheumatic disease of the wrist and arthritic hand positioning
- Bursitis, Tendonitis, Tenosynovitis
- Psychological assurance
- Post extensor tendon repair surgery
- Conservative management of keinbock's disease
- Sprains and strains

#### RECOMMENDATION:

- Use this product under strict guidance of a qualified doctor.

#### INSTRUCTIONS: [1]

- Discontinue use and seek guidance of a qualified doctor, in case of
  - Impaired sensation
  - Impaired blood flow
  - Allergy rash or itching
  - Pain increases or persists
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.