PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not iron.

Do not bleach.

Close hook loop fasteners before washing.

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

Batch code

Manufacturing date

Manufacturer

Catalogue number

Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JLPL, Mohali -160 055, (Pb) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-5288111

ISO
9001

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

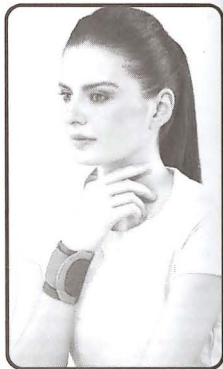
www.tynorindia.com

Wrist Wrap (Neoprene)
Wrist Brace with Double Lock

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Wrist Wrap (neoprene) & Wrist Brace with Double Lock

English

INTENDED USE :

Wrist Wrap (Neoprene) and Wrist brace with double lock are designed to support, protect and partially immobilize the wrist. They compress the area and retain the body heat to allay local pain and inflammation.

CONSTRUCTION:

Wrist Wrap (Neoprene) is made out of three layered Neoprene foam bonded fabric. The middle layer is made from elasticised neoprene rubber foam, which provides four way stretch to the fabric. The top layer is made out of a very special fabric with unbroken loops. Unbroken loop fabric offers the advantage of a very versatile closing system to provide easy application and a firm grip to the wrist.

Wrist brace with double lock is made from strong, porous orthopedic webbing. Double hook loop locking system enhances the immobilization and provides a firm grip to the wrist.

APPLICATIONS :

- Post operative and post cast care.
- Tendonitis, Tenosynovitis
- Arthritic care of the wrist

- Prophylaxis in exercise and games
- Carpal tunnel syndrome
- Sprain and Strain
- Dislocation of wrist joint

RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.

INSTRUCTIONS: (ⓘ)

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Allergy rash or itching
 - Pain increases or persists
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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